

Insul-Eze™

Frequently Asked Questions/ Troubleshooting



Q: What size vial works with the Insul-eze?

A: The Insul-eze works with most insulin vials and larger medicine vials.

Q; What syringes work with the Insul-eze?

A: Most 1 ml (100cc), 1/2 ml (50cc), 1/3 ml (30cc) and 1 ml tuberculin syringes work with the Insul-eze.

Q: How much does the Insul-eze magnify?

A: The Insul-eze magnifies all syringe calibrations two times for easy reading.

Q: Does the Insul-eze contain latex?

A: The Insul-eze does not contain latex. For a list of materials please contact AMBIMEDINC.

Q: Can you suggest an easier method to use the Insul-ese with one hand?

A: Gluing a magnet to the back of the Insul-eze and applying it to your refrigerator or other magnetized surface will help you to hold the device steady while you load your insulin syringe. Velcro* used in a similar manner may also be helpful.



908 Sir Francis Ave Capitola, CA 95010 · 831-475-1765 phone/fax
info@ambimedinc.com · www.ambimedinc.com

Insul-Eze is a registered trademark of AMBIMEDINC. All rights reserved.
*Other trademarks are property of their respective holders.