Instructions for the Insul-eze™

The Insul-eze is a one piece syringe loading device with built-in magnifier. Numbers on the disposable insulin syringe are magnified two times in the transverse direction. The Insul-eze is suitable for use with 1/4 cc (25 unit), 1/3 cc (30 unit), 1/2 cc (50 unit), and 1 cc (100 unit) syringes by Becton-Dickinson, Terumo, and most other manufacturers. The bottle receptacle holds securely all standard size insulin bottles.

Directions

1. Snap the bottle top into the receptacle at the end of the device (*Fig. 1*).
2. Rotate the Insul-eze so that the flat side is facing up (*Fig. 2*).
3. Hold a syringe as shown and place the needle into the long groove on the bottom of the device (*Fig. 3*). Press the syringe against the top of the groove.
4. Push the syringe toward the bottle until the needle pierces the bottle top (*Fig. 4*).
5. Push in the plunger so that some air is injected into the bottle (*Fig. 5*).
6. Turn over the Insul-eze so that the curved surface is facing up. Carefully rotate the syringe so that the magnified numbers are plainly visible (*Fig. 6*).
7. While holding the syringe securely, pull back on the plunger and withdraw the proper dose (*Fig. 7*).
8. The syringe is now ready for use and can be removed from the device (*Fig. 8*).

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