

Instructions for the ambi-Lancet® Eject

The ambi-Lancet is a reusable lancing device for the single user. It automatically injects the lancet into and retracts it from the fingertip to obtain a capillary blood sample for glucose monitoring or other tests that require one or two drops of blood. An adjustable tip allows the best depth of skin penetration for each individual user.

CAUTIONS:


- The ambi-Lancet is intended for the single user. To avoid exposure to contaminants, do not use the ambi-Lancet on multiple users.
- For safety and to prevent cross-contamination, always discard used disposable lancets in an appropriate sharps or biohazard container.
- The ambi-Lancet contains small parts and could be a choking hazard for children.

Instructions for use:

Clean your hands with soap and warm water; dry them thoroughly. Warming your hands will help in obtaining a blood sample. Follow the steps below.



1. Wash your hands with soap and warm water and dry them thoroughly. Remove the adjustable tip of the lancing device.

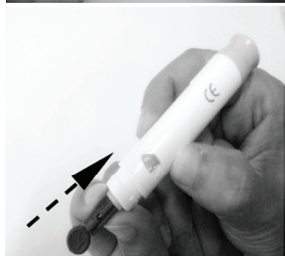

2. Insert a **new** lancet into the lancet carrier. Push down firmly until it is fully secured.



3. Remove the protective cap from the lancet and save it for later use. Replace the adjustable tip on the lancing device.



4. Choose the desired depth of penetration to get a good blood sample with the least amount of discomfort.
Adjustments:
0-1 soft or delicate skin
2-3 normal skin
4-5 thick or callused skin



Cleaning:


Use a clean damp cloth to wipe the outside of the ambi-Lancet. To avoid rusting the internal spring, do not submerge the device in water or wash the device in the dishwasher. The adjustable tip can be submerged in water after removing it from the device.

Warranty:


This AMBIMEDINC product carries a two (2) year warranty against manufacturers defects in material and workmanship and against breakage or failure under normal use. This warranty does not cover misuse or abuse.

- Contact AMBIMEDINC Customer Support at 831-475-1765 or info@ambimedinc.com or visit the website at www.ambimedinc.com for a list of compatible disposable lancets.


ambi-Lancet is a trademark of Ambi, Inc. All rights reserved.

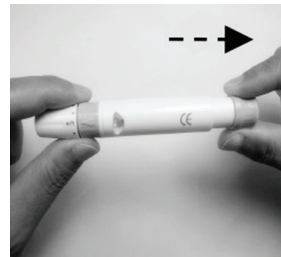


5. Hold the tip with one hand and pull the sleeve until it clicks. The indicator/trigger button will change color and indicate it is ready for use.

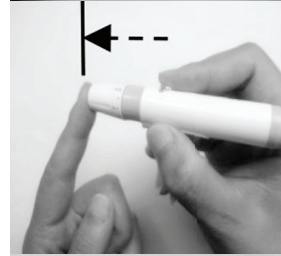

6. Hold the lancing device firmly against your finger. Press the trigger to puncture the skin. Wait a few seconds for a blood drop to form. Blood flow increases by gently massaging the finger.

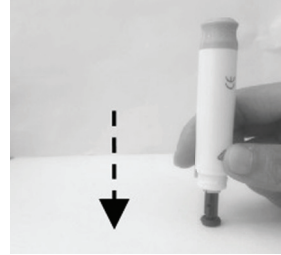

7. Remove the adjustable tip on the lancing device without touching the used lancet. When finished, put the protective cap back onto the exposed needle of the lancet.

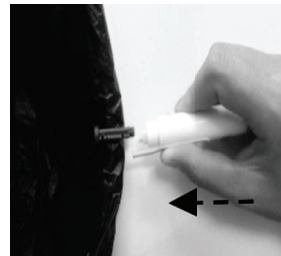

8. Point the lancing device away from you and slide the lancet ejector lever in a forward motion, disposing the lancet in an approved container. Return lancet ejector lever to its original position.

- **Do not reuse lancets.**